

## 3Shape Scanner Instructions (Suggested Workflow)

### Pre-Scan Instructions

Step  
**01**

Add "New patient", if applicable.

Step  
**02**

Select "New Case".

Step  
**03**

Select "Status Scan".

## Scan Instructions

### Step 01

Scan patient's lower and upper arches, and obtain a single occlusion scan as described in the [Proclaim Scanning Guide](#). It is recommended to skip the second occlusion. Manual alignment may be performed during occlusion scan, if needed.


### Step 02

Post-process case.

## Exporting STL files


### Step 01

Select the scan you would like to export, and press the 3 dots as shown to display a dropdown menu.

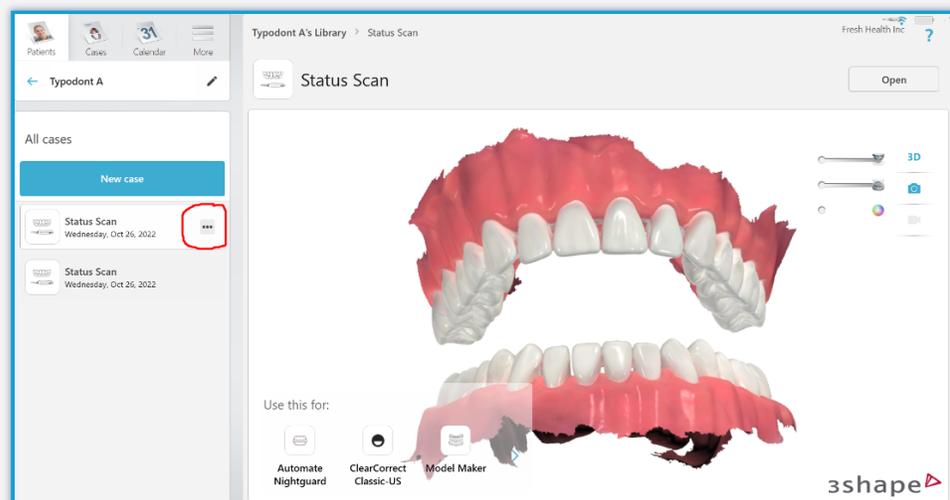


# Step 02

Select "Scans" and "Export".

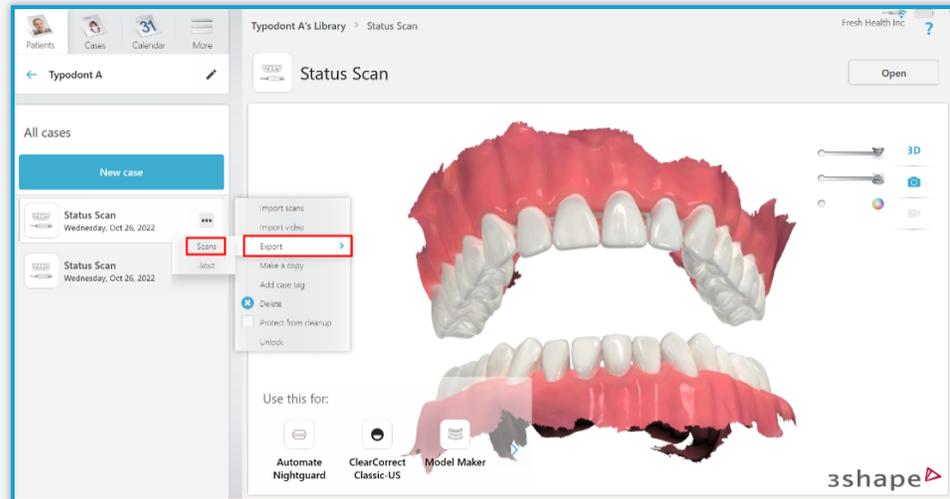


# Step 03

Select your desired destination folder. Ensure "STL" is selected under "Save as type".

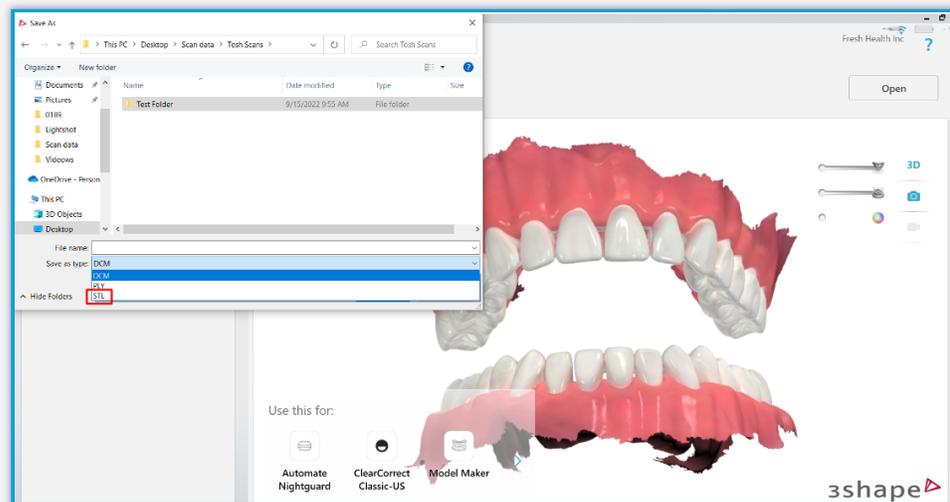


# Step 04

Label the files for exporting – this name will prefix each exported file.

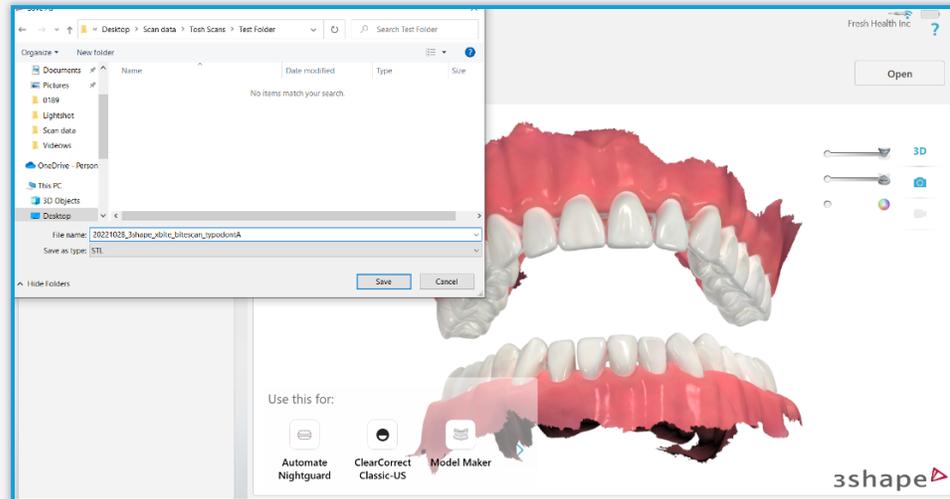

# Step 05

Files are exported. Note that each file has a suffix identifying the case.

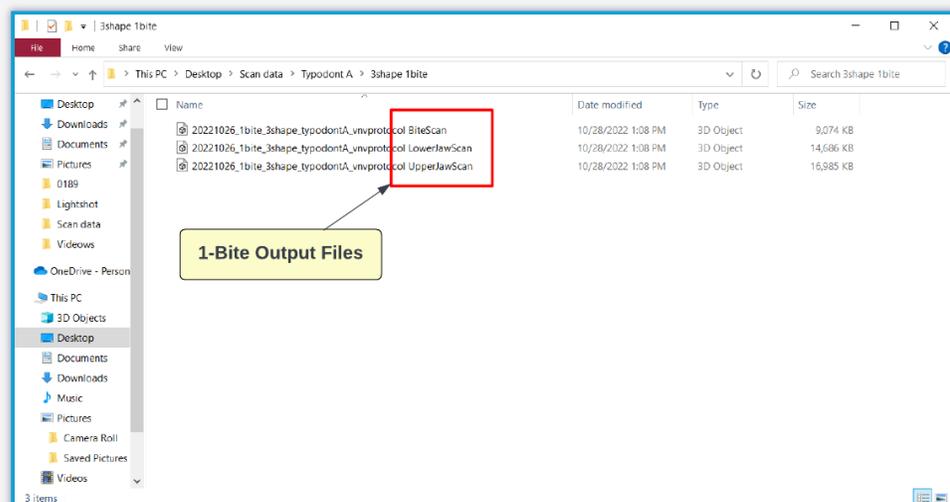

## Uploading to Proclaim Connect

### Step 01

Log into **Proclaim Connect** and follow **this link** for instructions on how to upload scans.