

PRE-SCAN INSTRUCTIONS

Step

1

Add **"New patient"**, if applicable.

Step

2

Select **"New Case"**.

Step

3

Select **"Status Scan"**.

SCAN INSTRUCTIONS

Step

1

Scan patient's lower and upper arches, and obtain a single occlusion scan as described in the **Proclaim Scanning Guide**. It is recommended to skip the second occlusion. Manual alignment may be performed during occlusion scan, if needed.

Step

2

Post-process case.

3Shape Scanner Instructions


(SUGGESTED WORKFLOW)

EXPORTING STL FILES

Step

1


Select the **scan you would like to export**, and press the **3 dots** as shown to display a dropdown menu.

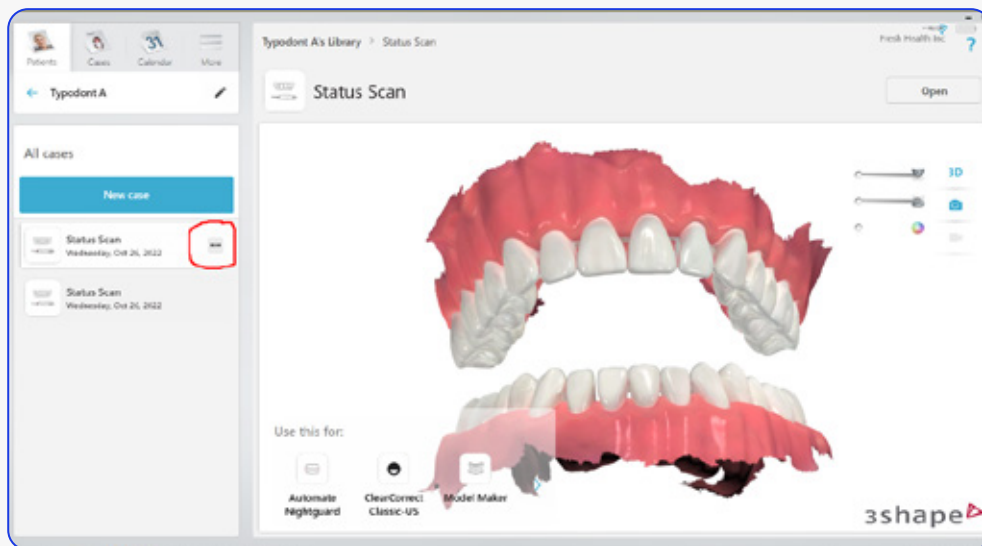

Step

2

Select **"Scans"** and **"Export"**.

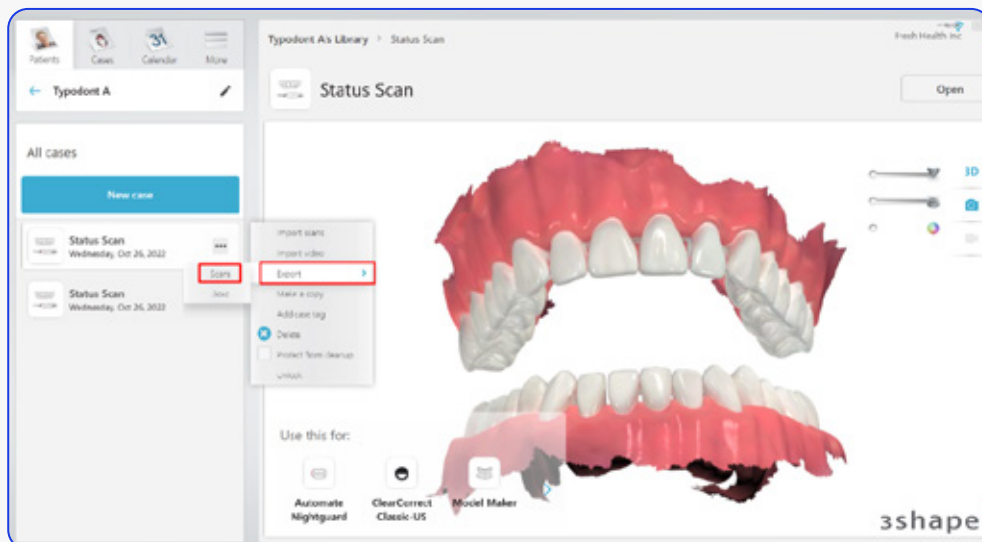


3Shape Scanner Instructions

(SUGGESTED WORKFLOW)

Step

3


Select your desired destination folder. Ensure **"STL"** is selected under **"Save as type"**.

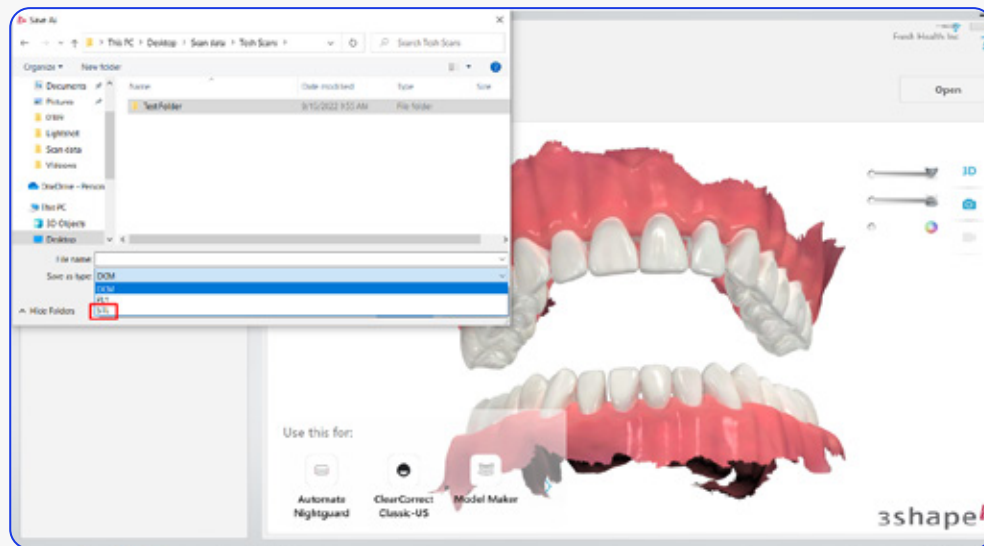

Step

4

Label the files for exporting – this name will prefix each exported file.

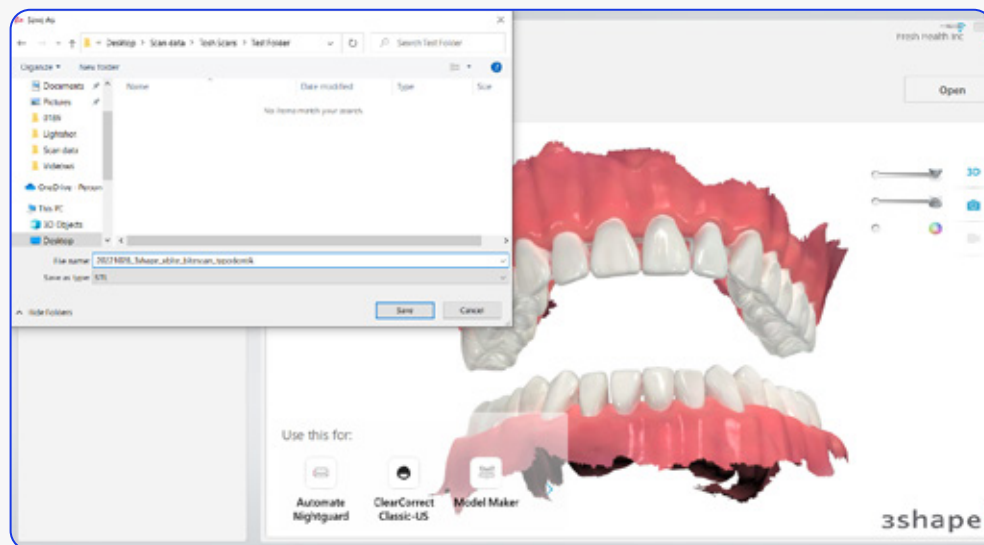


3Shape Scanner Instructions

(SUGGESTED WORKFLOW)

Step
5

Files are exported. Note that each file has a suffix identifying the case.

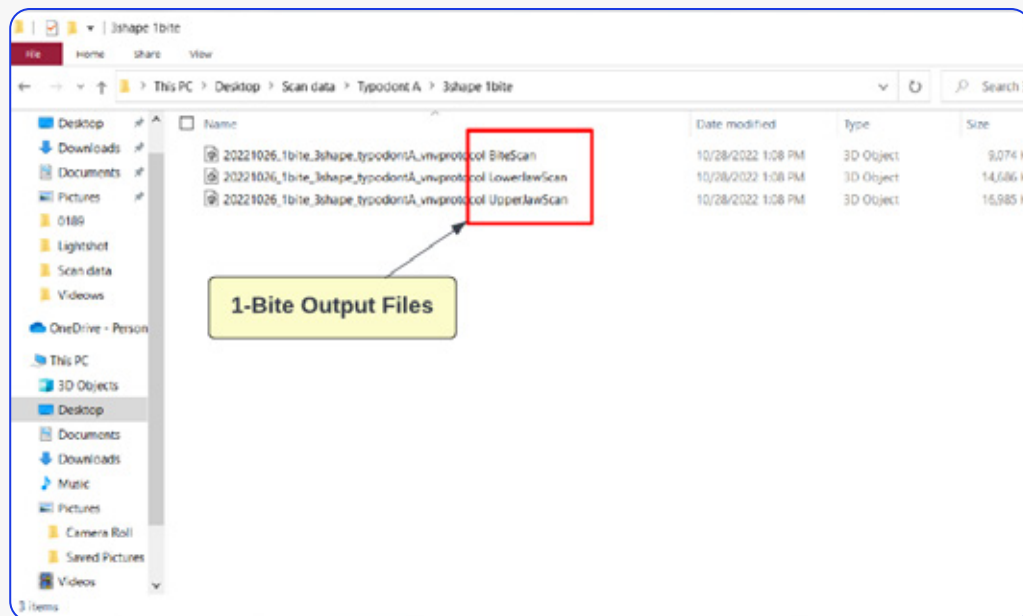

UPLOADING TO PROCLAIM CONNECT

Step
1

Log into [Proclaim Connect](#) and follow [this link](#) for instructions on how to upload scans.