

Intelliscan Scanner Instructions (Suggested Workflow for Proclaim)

Pre-Scan Instructions:

1. Add patient and initiate a new scan.

Scan Instructions:

1. Scan patient's lower and upper arches per standard scanning practice. Capturing additional gingival anatomy helps align the bite scan.
Note: To obtain scan data in the interproximal region, it may help to turn "AI" off.
2. For the bite scan, it is recommended to turn "AI" off. Obtain a single occlusion scan as described in the [Proclaim Scanning Guide](#), ensuring sufficient gingiva is scanned. It is recommended to skip the second occlusion. Manual alignment may be performed during occlusion scan, if needed.
3. Post-process case

Exporting STL files:

1. Select "Export".
2. In the pop-up window, select "STL" as the file option.
3. Select accept.
4. Three STL files will be exported for upload in Proclaim Connect.

