

True Definition Scanner Instructions

(SUGGESTED WORKFLOW)

PRE-SCAN INSTRUCTIONS

Step

1

Add a New Patient, and enter requested patient information.

Note: Use of scanning spray/powder may help to achieve better scan quality. Use of suction may also help to improve scan efficiency.

Step

2


Proceed to **"Scan"**.

SCAN INSTRUCTIONS

Step

1

Scan patient's lower and upper arches, and obtain a single occlusion scan as described in the Proclaim Scanning Guide. Only one occlusion scan is needed. Ensure arches are accurately aligned in the scan.

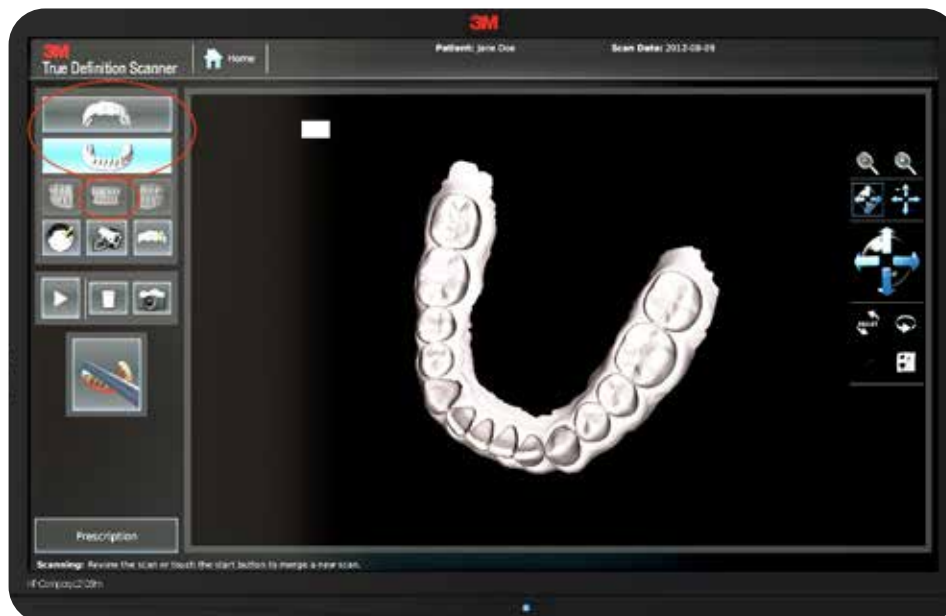

Note:

It is recommended to spray the shim with scanning spray/powder to improve scanning efficiency.

True Definition Scanner Instructions

(SUGGESTED WORKFLOW)

EXPORTING STL FILES

Step

1

After scanning, export files. Select the "Models and Appliances" option, and **select "OK"**.

Step

2

STL should be the only **file export option**.

UPLOADING TO PROCLAIM CONNECT

Step

1

Log into **Proclaim Connect** and follow **this link** for instructions on how to upload scans.