

PRE-SCAN INSTRUCTIONS

Step

1

Enter requested patient information on the scanner.

Step

2

Select **"Non-Restorative"** followed by **"Scan"**.

SCAN INSTRUCTIONS

Step

1

Scan patient's lower and upper arches, and obtain a single occlusion scan as described in the Proclaim Scanning Guide. Only one occlusion scan is needed. Ensure arches are accurately aligned in the scan.

EXPORTING STL FILES

Step


1

After scanning, **export files**.

Step

2

Select the patient folder, and select **"3D Model Export"** on the top right corner.

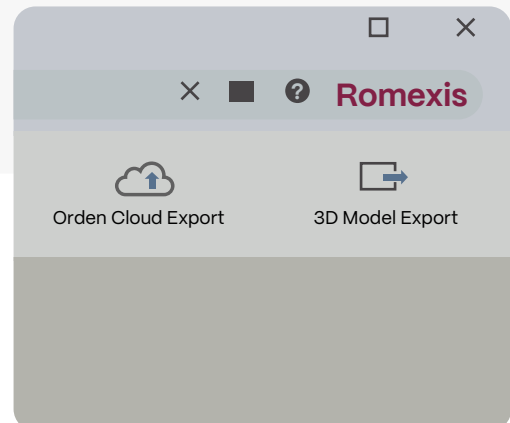

EXPORTING STL FILES

Step

3

Export in STL format. Three STL files should be exported.

UPLOADING TO PROCLAIM CONNECT

Step

1

Log into **Proclaim Connect** and follow [this link](#) for instructions on how to upload scans.